

GASTROENTEROLOGICAL DISEASE

Helicobacter Pylori Stool Antigen Rapid Test
Fecal Occult Blood Stool Antigen Rapid Test

Limit of detection



HP
4ng/mL

FOB
30ng/mL

CE


JD Biotech®
杏恩

**Helicobacter pylori Antigen and
Fecal Occult Blood Antigen Combo Rapid Test Kit**

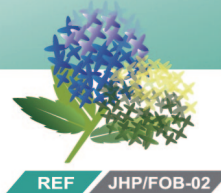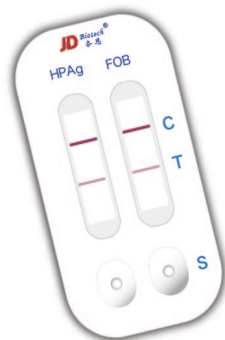

REF / JHP/FOB-02

1Test/Box

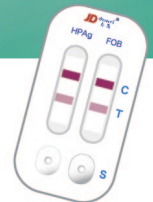

HOME TEST use: 1Test /Box
Professional use : 25 tests / Box

Qualitative & Quantitative assays are possible


Simple Assay Procedure

1 HP/FOB sample collection tube




It has 0.5ml sample buffer.

2 Collect sample

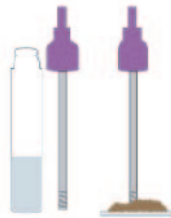


Take the sample collection swab, collect small amount at different site.

3 Mix well the sample and buffer

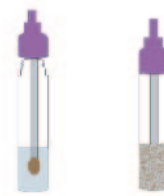

4 Break the head of sample collection tube

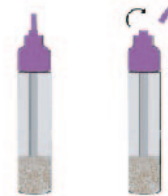

Break the head of purple cap.

5 Testing




Squeeze 3 to 4 drops of sample tube solution on the sample region S of HP and FOB

6 Wait 15 minutes



7 Read the result

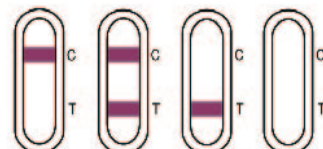

Neg. Pos. Invalid result



With Immuno-Gold Reader, it can be used for patient-follow up after treatment.

Jeil Daniel (JD) Biotech Corp.

Manufacture:

A201, 1st Building, No.69, Hua Yang Road,
Jinan, Shandong 250100 China

TEL: +86-531-82373551

FAX: +86-531-82373522

Taipei office:

4A-26, No.5, Sec.5, Hsin Yi Road,

Taipei City 110 Taiwan

TEL: +886-2-27253734 Fax: +886-2-27253844