

Sexual Transmitted Infections

Trichomonas Vaginalis

**Candida Albicans
(Yeast)**

Gardnerella Vaginalis



URINE SAMPLE SPECIMEN

TEST IN AN EASY WAY

- More than 1 million STIs are acquired everyday worldwide.
- STI diseases often are asymptomatic, and it represents a global public health problem and a major cause for morbidity and mortality.
- Not only HIV but also other STIs are influencing our life.
- Same-sex mating are at increased risk of contracting STIs.

SMART SCREENING



Simple Assay Procedure

1




Open the cap of sample buffer tube.

2




Pipete urine sample 0.5 ml.

3

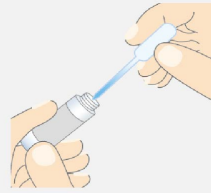

Add the urine sample 0.5 ml into the tube.

4

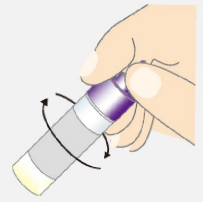


Swirl the tube clockwise for 5 seconds to mix well.

5




Take out the test card.

6

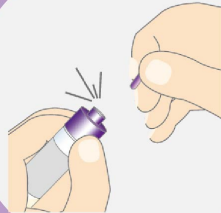

Break the top of sample buffer tube knot.

7

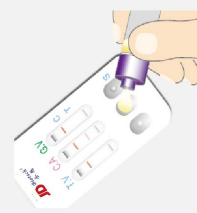

Add 4 drops of the urine mixture into card "S" region.

WAIT FOR 15 MINUTES





Over 95% Accuracy

Easy sample collection

Saving Cost

Saving Time

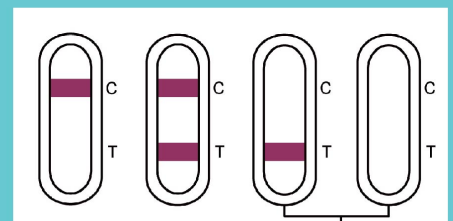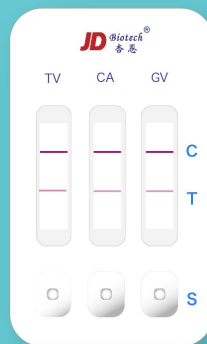

Neg. Pos. Invalid result

Je Daniel (JD) Biotech Corp.

 A201, 1st Building, No.69, Hua Yang Road,
Jinan, Shandong 250100 China
TEL: +86-531-82373551 FAX: +86-531-82373522
www.jdbiotech.com

Taipei office:
4A-26, No.5, Sec.5, Hsin Yi Road,
Taipei City 110 Taiwan
TEL: +886-2-27253734 Fax: +886-2-27253844

JD Biotech®
杏恩